

CutCure™ Antibacterial Wound Gel

Works Rapidly on
Minor Wounds of All
Shapes & Sizes



CutCure™ is a clear, antibacterial*, hemostatic gel that provides durable wound care for minor cuts and injuries during work-related mishaps and rugged activities. It works up to 17X faster than the standard of care of using a dry absorbent dressing alone†. CutCure™ gel contours and adheres to minor injuries, staying in place to deliver rapid bleeding control and rinses off easily- with no bandages to pull off. Made in USA and sourced from natural materials.



CutCuRe™

- 🔴 FDA Cleared Technology†
- 🔴 Creates a Durable Biopolymer Matrix Seal
- 🔴 Works Rapidly on Minor Wounds of All Shapes & Sizes
- 🔴 Convenient Single-Use Packets



79605 5pk

DIRECTIONS:

Apply enough gel to cover the surface area of the wound completely. Using a gauze pad, press and hold for approximately 30-60 seconds. Gel controls bleeding by adhering to the wound and sealing it. If gel does not adhere, gently wipe it off and apply new gel. Once the bleeding stops, clean around the wound. This area may be covered with a sterile bandage or non-stick dressing.

*Based on laboratory testing, product prevents growth of bacteria, like STAPH & MRSA, within the gel, FDA cleared under K172010, K180152, K182811, K192667

†Based on laboratory testing, product stops bleeding up to 17 times faster than using dry absorbent dressing alone.

First Aid + Safety + PPE + Safety Training + Mobile Testing + Fire Protection

UFAS-CCR_X_0424



Visit Us Online Today!

UniFirst.com/FirstAid // 800.869.6970

UniFirst +
FIRST AID + SAFETY